



polapaint

Paint-on tooth whitening system
(8% carbamide peroxide)

INSTRUCTIONS FOR USE

E
N
G
L
I
S
H



Pola Paint is an easy to use paint-on tooth whitening gel. It is a pleasantly flavored gel with a low viscosity which is easily applied onto the tooth surface. Before using Pola Paint, your dentist should assess your suitability for the product including your medical history and explain the application procedure.

INDICATIONS FOR USE:

- Whitening of discolored vital teeth
- Whitening of discolored non-vital teeth

INSTRUCTIONS:

1. Brush and floss teeth as normal.
2. Blot dry the front surface of teeth.
3. Whilst retracting the lips, apply a thin layer of Pola Paint gel to the front surfaces of the teeth, re-loading brush as required.
4. After application do not let your lips touch your teeth for 30 seconds to allow the gel to dry.
5. Leave Pola Paint gel on teeth for 30 minutes. Do not eat or drink during this time to maximize results.
6. At the end of 30 minutes, remove the Pola Paint gel by brushing with a tooth brush.
7. Use Pola Paint twice a day for 14 days.

For best results, one of the daily treatments should be applied before going to bed, allowing the Pola Paint to stay on overnight. When not in use, **firmly** replace cap to prevent gel from drying out.

COMPOSITION:

- carbamide peroxide
- ethanol
- glycerol
- additives
- water
- flavor

PRECAUTIONS:

To be used under the supervision of a dentist.

Do not use after expiry date.

Use at room temperature.

Pola paint should not be used by patients with known allergies or chemical sensitivity to peroxides, glycerol etc.

Do not smoke during or immediately after the whitening process.

Keep product away from heat and naked flame, as product is flammable.

Keep out of reach of children and pets.

Not to be used by pregnant or lactating women or children under the age of 14 years.

Discard Pola Paint after treatment has been completed.

MSDS available at www.sdi.com.au or contact your regional representative.

STORAGE:

Keep cool and out of direct sunlight at temperatures between 4°C - 20°C / 39°F - 68°F

DO NOT STORE IN CLOTHING POCKETS.

FIRST AID

Skin (contact): Pola paint may be irritating to skin. Wash with running water if contact occurs. If any soft oral tissue irritation occurs, remove gel, rinse mouth with salty water or sodium bicarbonate solution and drink milk.

Discontinue treatment and consult your dentist if there is persisting sensitivity or any reaction to the gel.

Eyes (contact): Open eye and flush thoroughly with water for 15 minutes.

Seek medical attention.

Made in Australia by SDI Limited

Bayswater, Victoria 3153

Australia 1 800 337 003

USA & Canada 1 800 228 5166

Brazil 0800 770 1735

Germany 0800 100 5759

France 00800 022 55 734

United Kingdom 00800 022 55 734

Spain 00800 022 55 734

Ireland 01 886 9570

Italy 800 780625

European Union Representative:

Southern Dental Industries Limited

Block 8, St Johns Court

Swords Road, Santry

Dublin 9, Ireland

www.sdi.com.au



4°-20°C
39°-68°F



PAK41107